

PICTERUS CARD HOLDER

INTRODUCTION

The Picterus Card Holder acts as a protective barrier between the Picterus® Calibration Card and a newborn's skin, preventing direct contact. Proper cleaning of the Picterus Card Holder between uses helps maintain hygiene and ensures safe and effective use of the Picterus® Calibration Card. Made of plastic, the Picterus Card Holder is compatible with alcohol-based cleaning solutions.



INSTRUCTIONS FOR USE

1. Insert the Picterus® Calibration Card into the holder, ensuring its edges slide under the small tabs on the holder. Do not touch the color patches.




2. Verify that the hole in the card holder aligns with the hole in the Picterus® Calibration Card.



3. Perform a Picterus Scan following the Instructions for Use for Picterus® Jaundice Pro.

4. After the scan is complete, slide the calibration card out of the holder.
5. Store the Picterus® Calibration Card in its original packaging for protection.

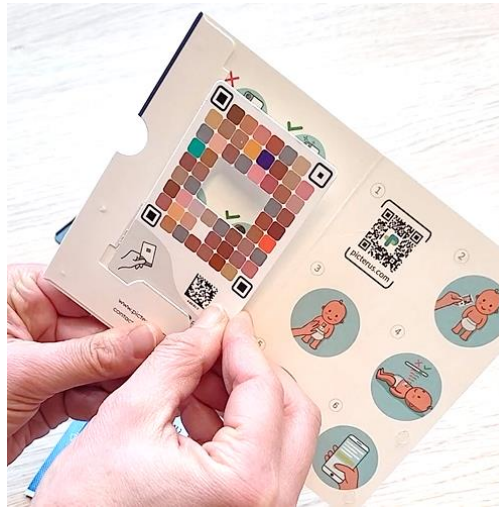

Note: For details regarding storage of the Picterus® Calibration Card, please refer to the Instructions for Use (IFU) of the Picterus® Jaundice Pro and the Picterus® Calibration Card. The card holder is designed to assist with handling and hygiene but does not replace the guidelines provided in the IFU.

6. Clean the back of the holder with an alcohol-based solution or wipe (recommended: isopropyl alcohol 70-90%, or ethanol 60-80%).
7. Allow the holder to dry completely before reinserting the Picterus® Calibration Card.